

Medical device**Dear Patient,**

Please read these instructions for use carefully because they contain important information about what you need to know when using this medical device. If you have any questions, please ask your doctor or pharmacist.

INSTRUCTIONS FOR USE

Tyrosur[®]

CareExpert Wound Gel

What Tyrosur[®] CareExpert Wound Gel is and what it is used for

Tyrosur[®] CareExpert Wound Gel is a hydrogel that complies with the principle of moist wound treatment. It thus supports the natural healing process in superficial skin lesions as well as in light burns (e.g. sunburn).

Mechanism of action

Moist wound healing is a long-established principle. The hydrogel regulates the moisture in the wound. Excess wound secretions are drawn away, and the wound is kept moist, favouring the healing process.

When must Tyrosur[®] CareExpert Wound Gel not be used?

Tyrosur[®] CareExpert Wound Gel must not be used in the event of hypersensitivity (allergy) to any of the ingredients.

When is particular caution required with the use of Tyrosur[®] CareExpert?

When used near the eyes: the gel must not get into the eyes, as this may lead to burning.

Can Tyrosur[®] CareExpert Wound Gel be used during pregnancy and breast-feeding?

As a general rule, you should always consult a doctor before using products during pregnancy and breast-feeding, regardless of whether they are ingested or applied to the skin.

How to use Tyrosur[®] CareExpert Wound Gel

Apply Tyrosur[®] CareExpert Wound Gel to the affected areas of skin several times a day.

How long should Tyrosur[®] CareExpert Wound Gel be used?

Tyrosur[®] CareExpert Wound Gel is intended to support the natural healing process. If your symptoms get worse or if no improvement is seen, please contact your doctor or pharmacist.

What are the side effects of Tyrosur[®] CareExpert Wound Gel?

The gel contains propylene glycol and can therefore cause skin irritation. In rare cases, hypersensitivity (allergic) reactions may occur.

What else do I need to know?

The expiry date is printed on the carton and the tube. Do not use this pack after this date. Do not store the gel above 25°C. Do not use for longer than 6 months after the first use.

What are the ingredients of Tyrosur® CareExpert Wound Gel?

The Gel contains carbomer, dexpanthenol, allantoin, vitamin E, trometamol, propylene glycol, cetylpyridinium chloride, ethanol 96%, purified water, citric acid, xanthan gum.

What pack sizes are available?

Pack sizes: packs with 25 g, 50 g and 100 g of gel.

Manufacturer / Distributor in Germany

Engelhard Arzneimittel GmbH & Co. KG
Herzbergstr. 3
61138 Niederdorfelden
Telephone: +49 6101 / 539 - 300
Telefax: +49 6101 / 539 - 315
Website: <http://www.engelhard.de>
e-mail: info@engelhard.de

**ENGELHARD****Last updated**

May 2019

CE
0297



Keep out of the reach and sight of children.